

Biocard HCG

A pregnancy test for home use.


Rapid, easy-to-use and reliable pregnancy test which can detect pregnancy hormone from your urine on the first day of the missed menstrual period

Read the instruction for use carefully before beginning the testing. Besides the test you need only a watch for timekeeping. Perform the test in a well-illuminated place.

TEST PROCEDURE



1. Open the pouch protecting the Biocard HCG test just before use. Remove the test from the pouch.

TEST WINDOW

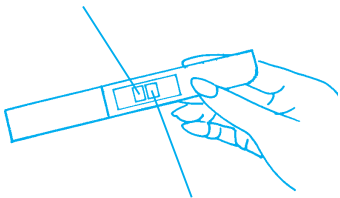


CONTROL WINDOW

2. Remove the cap off the sample end of the test device. Turn the test so that the perforated side of the sample end is facing you, as shown below.

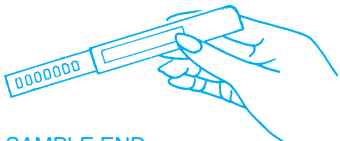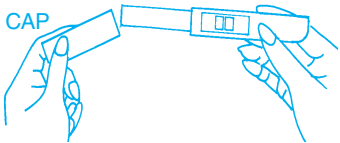


SAMPLE END

3. Hold the test as shown in the drawing below and put the perforated sample end of the device under urine stream for a few seconds.

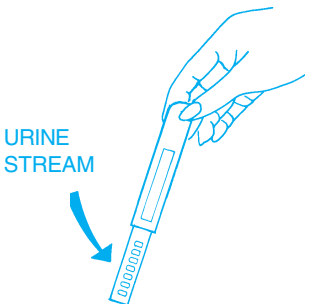

4. Withdraw the device from the urine stream and keep the sample end of the device pointing down for some 10-15 seconds to let extra drops of urine fall down. (Do not turn the sample end of the test device up at any time before reading the result.)

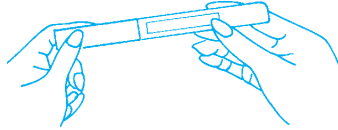

5. Replace the cap on the device and place the device on a flat surface with the window side up. Read the result 5 minutes later in well-illuminated place.

INTERPRETATION OF THE TEST RESULT

You are pregnant if a red line appears both in the test and the control indicator window. The colour of the test line may vary from faint to dark depending on the amount of hCG hormone contained in the urine sample. The test line must be consistent with the control line and appear vertically. Do not compare intensity of the colour in the test and control window.



PREGNANT

You are not pregnant if a coloured line appears only in the control indicator window.



NON PREGNANT

If the test result is unsettled or difficult to read after the 5 minute testing time, wait for 5 more minutes and read the result once again. In case the test result remains extremely faint, which indicates that the amount of hormone in your urine is very low, repeat the testing with a new test device 2 days later. If control line is not formed you have likely not followed the instruction for use or the test unit is damaged, in such a case repeat the testing with a new test unit.

Manufacturer:
ANI BIOTECH OY,
Tiilitie 3,
01720 Vantaa,
Finland
For more information, send e-mail to
customer.service@anibiotech.fi

NOTES AND LIMITS

Biocard HCG is only intended for the detection of hCG hormone in urine samples in accordance with the instructions for use. If the instructions for use are not followed in detail, outcome of the test may be false.

Use each Biocard HCG test unit only once.

Do not use expired tests or tests with a damaged pouch. If the test result is unclear or contrary to your expectations, run a new test 2 days later.

The test can be discarded as ordinary household waste.

OUR EXPERT GIVES AN ANSWER TO A FEW COMMON QUESTIONS ABOUT PREGNANCY

How does the Biocard HCG test function?

When you become pregnant, your body starts to produce hCG hormone, which can be indicated with Biocard HCG .

How early can I use Biocard HCG?

Biocard HCG is very sensitive. The test can be used already on the first day of a missed menstrual period.

At what time of the day can I run a Biocard HCG test?

You can use the Biocard HCG test at any time of the day.

To which extent can I rely on the result given by a Biocard HCG test?

You can rely on the result given by the Biocard HCG test, if you carefully follow the instructions for use. The test may render a false positive result only in a few specific medical cases. If the test result is contrary to your expectations, consult your doctor.

Can alcohol or medicines affect the test result?

Only medicines containing hCG hormone can have an effect on the test result. In such cases, the most suitable testing time should be determined together with your doctor. Alcohol does not influence the test result.

How long does the test result remain visible?

The positive test result remains unchanged for several hours.

What should I do when the test indicates that I am pregnant or if I am not sure of the test result?

Consult your doctor or a health clinic.