


# BIOCARD™ CANDIDA

Latex agglutination test

**Test cards for rapid detection of Candida antigen in vaginal swab specimens of symptomatic patients**

Cat. no. 3-007-000

- \* Direct detection of Candida antigens in vaginal swab specimens
- \* Easy and quick test procedure
- \* Room temperature storage
- \* 32 tests per package
- \* Sensitivity and specificity on other side of paper
- CE – marked

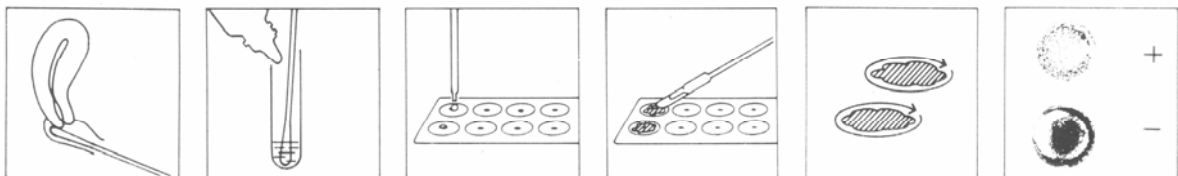

Vantaa, Finland

Tel. +358 - 20 155 7510

Fax. +358 - 20 155 7517

[www.anibiotech.fi](http://www.anibiotech.fi)

# BIOCARD™ CANDIDA

## Latex agglutination test

The overall accuracy of each method as compared to ultimate diagnosis.

Diagnosis	Quantity	Biocard Candida		Latex Test		Culture	
		+	-	+	-	+	-
Positive for Vulvovaginal Candidiasis	60 *	55	5	53	7	58	2
Negative for Vulvovaginal Candidiasis	330	9	321	10	320	40	290
Sensitivity	-	91.7 %		88.3 %		96.7 %	
Specificity	-	97.3 %		96.9 %		87.9 %	
Predictive Value of a Positive Result	-	85.9 %		84.1 %		59.2 %	
Predictive Value of a Negative Result	-	98.5 %		97.9 %		99.3 %	
Efficiency	-	96.4 %		95.6 %		89.2 %	

\* Includes 2 specimens positive for Candida by microscopy, but not demonstrated by culture.